


## TRANSPORTMEDIUM (STUART'S)

### Instructions for use

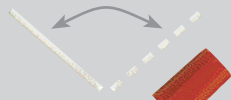

1.  
After specimen collection  
place the swab in the  
tube. The tip has to reach  
the bottom.

2.  
Break the swab forward  
and back at the top of  
the glass tube.



3.  
When the screw cap is  
screwed on, the swab is  
fastened in the cap.



4.  
Place the tube in the  
transport container.

1. edition, January 2003



STATENS  
SERUM  
INSTITUT

5 Artillerivej  
2300 Cph. S  
Denmark

